

# ZOLL<sup>®</sup>

# M Series<sup>®</sup> CCT

## CONFIGURATION GUIDE



An issue or revision date for this manual is shown on the front cover.

If more than three years have elapsed since this date, contact ZOLL Medical Corporation to determine if additional product information updates are available.

ZOLL, M Series, and *stat padz* are registered trademarks, and PD, PowerCharger, Connect Alarm, ZOLL Data Control Software, Smart Alarms, *Speed Pack*, and Preconnect are trademarks of ZOLL Medical Corporation.

12SL and Catalyst MUSE are trademarks of GE Medical Systems.

Copyright 2006 by ZOLL Medical Corporation. All rights reserved.

---

INTRODUCTION .....	1
CONFIGURING THE UNIT .....	1
SELECT LANGUAGE .....	2
MAIN CONFIGURATION MENU .....	3
PRINT CONFIG .....	3
RESTORE DEFAULTS .....	3
READ FROM CARD .....	4
CHANGE CONFIG .....	5
CONFIGURATION SETTINGS .....	7
NOTCH FILTER .....	7
ALLOW CARD ERASE .....	7
DISPLAY ELAPSED TIME .....	7
VOICE MARKERS ENABLED .....	7
SET REPORT RESTART DELAYS .....	8
PRINT 12 LEAD 4x3 .....	8
AUTO SELF-TEST TIMEOUT PERIOD .....	8
ASYNC SOFTKEY IN PACE MODE .....	8
CODE MARKERS .....	8
DEVICE IDENTIFIER .....	10
MANUAL MODE PASSCODE (ADVANCED AED MODELS) .....	11
CONFIGURATION MODE PASSCODE .....	11
ENERGY LEVEL: SHOCK 1, 2, 3 .....	12
AUTO ANALYZE 3 TIMES .....	12
"NO SHOCK ADVISED" PROMPT .....	12
"CHECK PATIENT" PROMPT .....	12
DISPLAY DO CPR MESSAGE .....	12
DISPLAY ECG IN SEMI-AUTO MODE .....	12
STORE TO CARD IN AUTO MONITOR MODE (AED MODELS) .....	13
AUTO CHARGE IN ADVISORY MODE .....	13
RECORD TO CARD IN ADVISORY MODE .....	13
ENABLE VOICE IN ADVISORY MODE .....	13
DISPLAY HEART RATE IN SEMI-AUTO MODE .....	13
DISPLAY LEAD ON POWER-UP .....	13
FREQUENCY RESPONSE .....	13
ENABLE 12-LEAD IN SEMI-AUTO MODE (AED WITH 12-LEAD) .....	13
QRS VOLUME ON POWER-UP .....	13
PACE RATE SETTING ON POWER-UP .....	14
DISPLAY IMPLANTED PACE SPIKES .....	14
GAIN ON POWER-UP .....	14
RETAIN SYNC AFTER DEFIB .....	14
AUTO GENERATE STRIPS .....	14
PRINT 3 LEADS WHEN LEADS SEL .....	14
REPORT MEMORY CARD ERRORS .....	14
AUTO ANALYZE ON POWER UP (AED SEMI-AUTOMATIC MODE) .....	14
AUTO TRANSMIT AFTER 12 LEAD .....	14
FAX/COMMUNICATION PHONE NUMBERS .....	15
12 LEAD ANALYSIS .....	17
12 LEAD PRINTOUT .....	17
PRINT 12 LEAD MEASUREMENTS .....	17
12 LEAD FAX FORMAT .....	17
PRINT 2 COPIES 12 LEAD .....	17
SET LEAD GROUPS .....	18
12 LEAD FREQ RESPONSE .....	18

---

LEAD GROUP DEFAULT .....	18
PRINT 12 LEAD INTERPRETATION .....	18
DEFIB DEFAULT TO PADS .....	18
CELL PHONE TYPE .....	19
ETCO2 OPTIONS .....	19
ENABLE LEADS OFF .....	19
BLOOD PRESSURE UNITS .....	19
NIBP AUTO INTERVAL DEFAULT .....	19
TRIGGER NIBP MEAS ON BP ALARM .....	19
TRIGGER NIBP MEAS ON HR ALARM .....	19
NIBP FIRST INFLATE PRESSURE DEFAULT .....	19
BASIC AUTO ENERGY ESCALATION .....	20
MUSE SITE AND LOCATION .....	21
12 LEAD SECS PER LEAD .....	21
12 LEAD PRINT SPEED .....	21
UPLOAD BAUD RATE .....	21
TRENDING REPORT ZOOM LEVEL .....	21
TRENDING ENABLED .....	22
AED DEFIB READY HOLD TIME (AED SEMI-AUTOMATIC MODE) .....	22
BATTERY TYPE USED .....	22
ALARM LIMITS AT POWER UP .....	23
ALARMS ACTIVE AT POWER UP .....	23
LOW BATTERY TONE .....	23
CLOCK SYNCHRONIZATION .....	23
DAYLIGHT SAVINGS .....	24
CARD CAPACITY MESSAGE ENABLED .....	24
ERASE CARD PROMPT AT PWR OFF .....	24
REMOVE CARD PROMPT AT PWR OFF .....	25
SYNC DEFIB EXCLUDED .....	25
PTCA SETTINGS .....	25
POWER UP PATIENT TYPE (NIBP) .....	25
TEMPERATURE UNITS .....	26
DISPLAY COLORS .....	26
POWER UP TRACE SELECTION .....	26
IBP1 LABEL AT POWER UP .....	27
IBP2 LABEL AT POWER UP .....	27
IBP1 RANGE AT POWER UP .....	27
IBP2 RANGE AT POWER UP .....	27
CONFIGURATION OPTION TABLES .....	29
GENERAL .....	29
ADVISORY AND SEMI-AUTOMATIC MODES .....	31
MANUAL MODE .....	32
ETCO2 OPTION .....	33
NIBP OPTION .....	33
PTCA SETTINGS OPTION .....	33
EXPANDED DISPLAY OPTION .....	33
IBP OPTION .....	34
TEMPERATURE OPTION .....	34

# Introduction

The M Series CCT is designed with several user selectable features that allow operators to configure the device according to their protocols and local requirements. This manual describes the configuration options and how to change them. This manual also lists the configurable features, options and default settings. Some features are available to manual mode operators only.


The default settings shown below are for North American units only. To print the current configuration of your unit, while the unit is in the configuration mode, press the **Print Config** softkey.

## CAUTION

The following features are not currently implemented on the M Series CCT: Voice Markers and Auto Self-Test.

# Configuring the Unit

After the unit has been off for 10 seconds or more, simultaneously press and hold the rightmost and leftmost softkeys. Turn the selector switch to **MONITOR** or **ON** while holding the buttons depressed for approximately four (4) seconds. After four (4) seconds the following screen will appear.

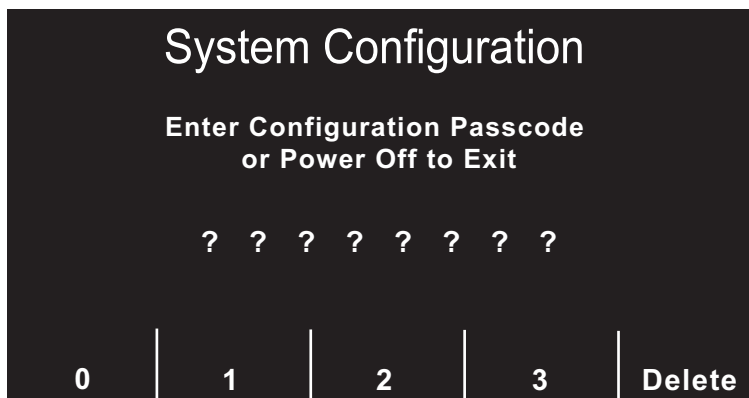

Users entering System Configuration for the first time must enter the default passcode (00000000) for System Configuration to continue.

Press the **0** (zero) softkey. This moves the highlight to the next digit. Repeatedly press the **0** (zero) softkey until all eight digits have been entered. The Configuration Passcode screen will then automatically advance to the Select Language menu.

To change one of the entered digits use the **Delete** softkey to highlight the incorrect digit and reenter the correct code.

## Select Language




The first configurable option displayed after entering the correct pass code is the Select Language option. Press the **Prev Value** and **Next Value** softkeys to move the highlight from one language to the other.

Some M Series CCT units include software that allows the device to operate in either of two languages. If the languages displayed do not meet your needs, contact ZOLL Technical Service for a language upgrade.

**Caution:** Print out the summary report before changing languages. If a summary report containing Code Marker events is printed after changing languages, the Code Marker events will not match.

Press the **Enter** softkey to select the highlighted language and move to the Main Configuration menu. All subsequent configuration menus will be in the selected language. Press the **Exit Config** softkey to display the following three choices:

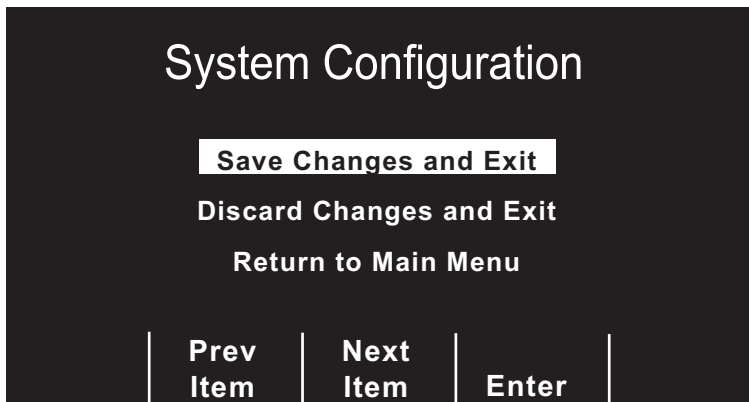


Press the **Prev Item** or **Next Item** softkey to scroll the highlight to the desired function. Press the **Enter** softkey to activate the selected function.

If the Return to Main Menu is selected, the unit will then display the Select Language menu described in the previous section.

## Main Configuration Menu

The Main Configuration menu is displayed whenever the **Enter** softkey is depressed from the Language Select menu or the **Main Menu** softkey is depressed while displaying other system configuration screens.

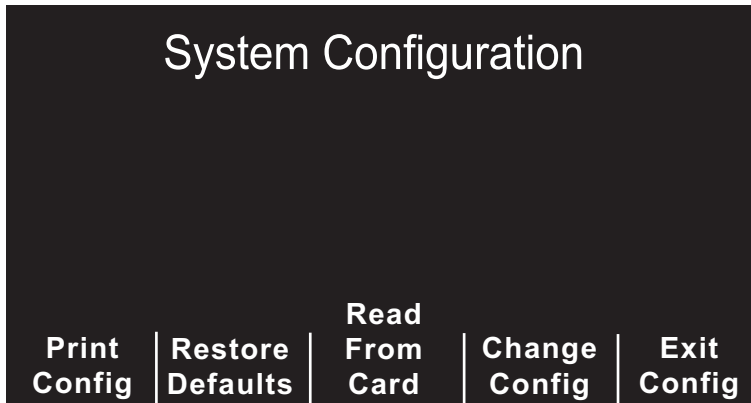


This menu provides the following five (5) softkey options: **Print Config**, **Restore Defaults**, **Read From Card**, **Change Config**, and **Exit Config**.

### Print Config

Pressing the **Print Config** softkey will cause the unit to print the current unit configuration settings.


### Restore Defaults

Pressing the **Restore Defaults** softkey causes the unit to reconfigure to its default settings. (See the "Configuration Option Tables" on page 29). The display language is the only setting that will not revert to default. Once the default settings have been restored, the "Defaults Restored" and "Press Change Config to View" messages are displayed.

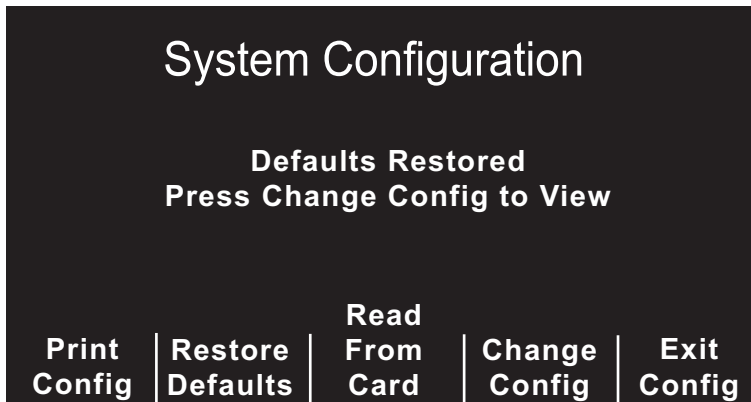


## Read From Card

Pressing the **Read From Card** softkey prompts the user to choose either a language or configuration card. The following screen will be displayed.

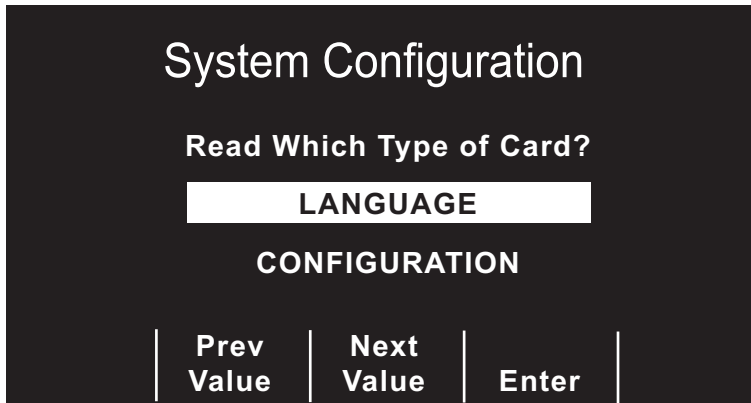

To move the highlighted area from LANGUAGE to CONFIGURATION and vice versa, press the **Prev Item** and **Next Item** softkeys. Press the **Enter** softkey to begin reading and storing the language settings or configuration settings from the user installed memory card.

If a language card has been read and loaded, the "Card Read" message displays. If a configuration card has been read and loaded, the "Card Read" and "Press Change Config to View" messages display (see below):

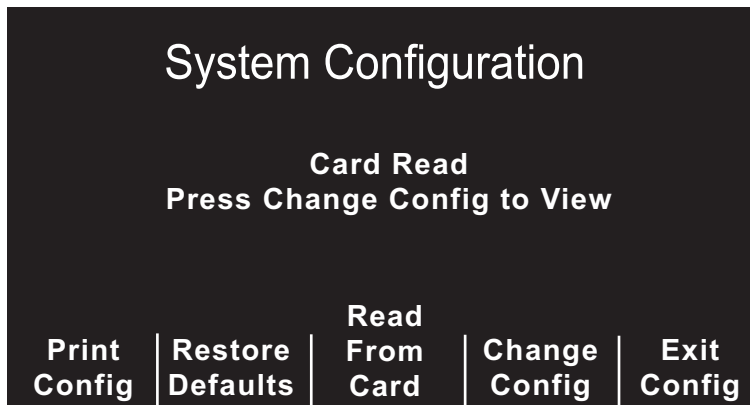

If there is an error reading the memory card, one of the following error messages will be displayed on the monitor.

EROR MESSAGE	DEFINITION
CARD READ FAILED	No response from card reader
BAD CARD	Cannot read from card
WRONG CARD TYPE	Card is the wrong type
CONFIG DATA NOT COMPATIBLE	Internal software is unable to read card data
NO CARD INSERTED	Card is not installed in PCMCIA slot
CARD EMPTY	No data on card



## Change Config

Pressing the **Change Config** softkey causes the unit to display the first configuration option menu.

There are three user configurable features contained within each option menu. (Refer to the “Configuration Option Tables” on page 29 for all available options and default settings).

(This page has intentionally been left blank.)

# Configuration Settings

System Configuration				
NOTCH FILTER		60Hz		
ALLOW CARD ERASE		Yes		
DISPLAY ELAPSED TIME		No		
Change Value	Prev Item	Next Item	Main Menu	Exit Config

The name of each configurable feature is displayed in the left column of the option menu and that feature's current setting is shown in the right column.

Pressing the **Change Value** softkey allows you to sequence through the available settings for the highlighted feature. When the desired setting is displayed, pressing the **Next Item** softkey will enter the selection and move the highlight to the next feature. Pressing the **Prev Item** softkey will enter the selected setting and move the highlight to the previously selected feature. Pressing the **Main Menu** softkey will cause the unit to return to the Main Configuration menu. Pressing the **Exit Config** softkey will cause the unit to display the "Save Changes and Exit" screen.

NEXT MENU is displayed on the screen when other option menus are available. Press the **Next Item** softkey until the new menu is displayed.

PREV MENU is displayed on the screen when other previously displayed option menus are available. Press the **Prev Item** softkey until the previous menu is displayed.

## Notch Filter

Sets the ECG Notch Filter frequency for proper AC mains interference rejection.

## Allow Card Erase

Allows the user to erase PCMCIA data cards on the M Series CCT unit when set to "Yes".

## Display Elapsed Time

Allows the user to display the elapsed time since the unit was turned on, when enabled. The elapsed time will continue to be counted for up to 10 seconds after power down. This will give the operator adequate time to change the battery without resetting the elapsed timer.

Elapsed time will be reset to zero (0:00) whenever the unit has been off for more than 10 seconds.

## Voice Markers Enabled

When voice markers are enabled, the unit will record five seconds of audio data in internal memory immediately following each depression of the **CODE MARKER** button. This audio recording along with a time stamp can be played back when the **SUMMARY** button is pressed.

**Note:** This option is not yet implemented.

## Set Report Restart Delays

Allows review of, or changes to the length of time the device must be turned off before it erases all data stored in either summary report memory or trend memory.

With Set Report Restart Delay highlighted and Set/Review selected, press the **Change Value** softkey. The following screen displays:

The screenshot shows a dark background with white text. At the top, it says "System Configuration" in a large font, followed by "RESTART DELAY TIMES" in a smaller font. Below this, there are two rows of settings. The first row is "SUMMARY REPORT RESTART DELAY" with a value of "15 MIN" to its right. The second row is "TREND REPORT RESTART DELAY" with a value of "15 MIN" to its right. At the bottom of the screen, there are four softkey labels: "Change Value", "Prev Item", "Next Item", and "Enter", each separated by a vertical line.

Pressing the **Change Value** softkey will scroll the highlight through the available delay times.

Pressing the **Prev Item** and **Next Item** softkeys will scroll the highlight to the next or previous restart delay feature.

Pressing the **Enter** softkey will cause the unit to return to the main configuration menu.

## Print 12 Lead 4x3

Determines how many copies of a 12-lead 4x3 report will print after pressing and holding the **RECORDER** button for three seconds. When set to "1 Copy" or "2 Copies" one or two copies of the 12-lead 4x3 report will print. When set to "None", pressing and holding the **RECORDER** button will cause a one or three lead DIAG BANDWIDTH ECG strip to be printed.

## Auto Self-Test Timeout Period

Allows the user to set the time between tests or disable the automatic tests. The unit is capable of automatically running a self-test at seven to thirty day intervals. If the unit is off, plugged in, and the test connector is attached to the Multi-Function cable, the unit will test the battery status, charge and discharge the defibrillator, measure the amplitude and timing of the pacing waveform, and measure the impedance of the test port.

**Note:** This option is not yet implemented.

## Async Softkey in Pace Mode


Sets the unit to display the Async Pacing On/Off softkey when the unit is in pace mode.

## Code Markers

Allows review of or changes to the preconfigured list of Code Markers available for use in PACER, MONITOR, and DEFIB modes.

**Note:** MONITOR Code Markers are not used in AED Models.


With Code Markers highlighted and Config/Review selected, press the **Change Value** softkey. The following screen displays:

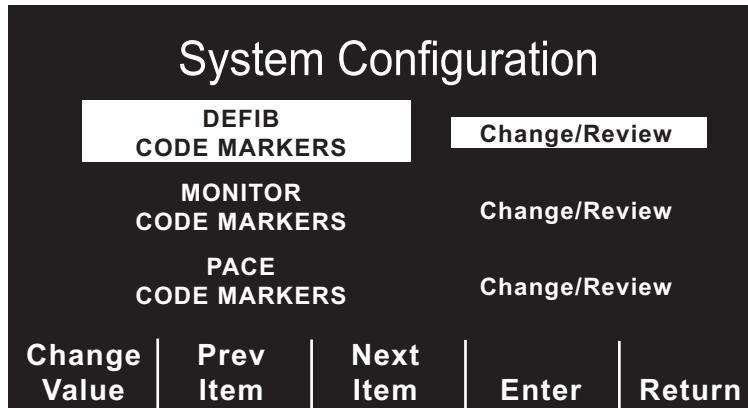

Pressing the **Change Value** softkey will change the highlighted setting from Change/Review to Set to Defaults and vice versa.

Pressing the **Prev Item** and **Next Item** softkeys will scroll the highlight to the next or previously highlighted code markers feature.

Pressing the **Enter** softkey when the Change/Review option is highlighted for one of the code markers features causes the following screen to be displayed (this example shows the Defib Code Markers screen).

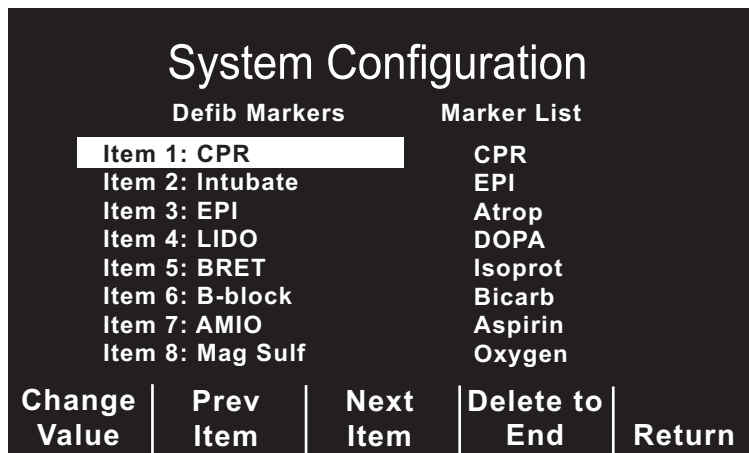

Pressing the **Change Value** softkey will highlight the first clinical action in the Marker List column and change the softkeys so that a new clinical action may be entered (shown below).

Pressing the **Prev Marker** and **Next Marker** softkeys scroll the highlight through the entire Marker List. Press the **Enter Marker** softkey to replace the highlighted item in the Defib markers column with the highlighted clinical action in the Marker List.

Pressing the **Prev Item** or **Next Item** softkeys will scroll the highlight to a different item number in the Defib Markers column.

Pressing the **Delete to End** softkey will delete all configured Defib markers from the highlighted item number to the end of the list. The user can then enter new clinical actions for each item.

Pressing the **Return** softkey will return the unit to the Change/Review Code Markers display where the user can choose Defib, Monitor or Pace Code Markers.


Pressing the **Return** softkey again causes the unit to return to the previous menu.

## Device Identifier

Sets an eleven-digit device identifier code. This code is divided into two sub codes (Site ID, eight alphanumeric characters, and Device ID, three alphanumeric characters). The device identifier is printed on the 12-lead strip chart, summary report, fax and Catalyst™ MUSE® printouts.

**Note:** If you are transmitting 12-lead ECG data to a Catalyst MUSE system, it is recommended that you set up the device identifier.

Pressing the **Change Value** softkey causes the following screen to display:

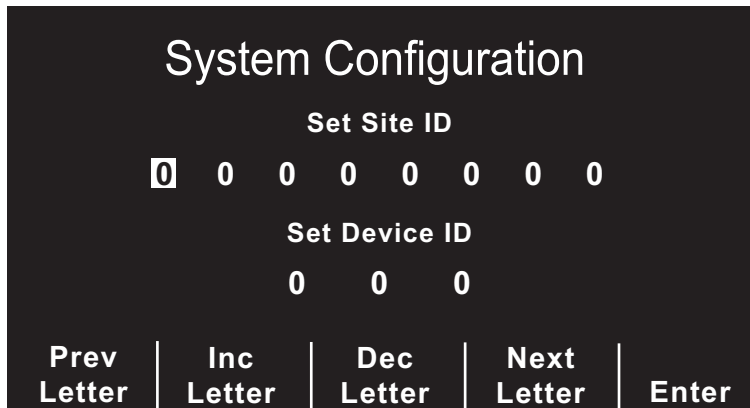

Pressing the **Prev Letter** softkey moves the highlight left to the previous entered digit.

Pressing the **Inc Letter** or **Dec Letter** softkeys increments or decrements the selected digit.

Pressing the **Next Letter** softkey moves the highlight right to the next digit to be entered.


Pressing the **Enter** softkey selects the values that have been entered and returns to main configuration menu.

## Manual Mode Passcode (Advanced AED Models)

Allows the three-digit Manual Mode Passcode to be set or changed. Each digit entered can only be set between 0 (zero) and 3 (three) inclusive.

Entering “000” as the pass code disables password protection of the Manual Mode. When “000” is selected, no password is required to enter the Manual Defibrillation Mode.

Pressing the **Change Value** softkey causes the following screen to display:

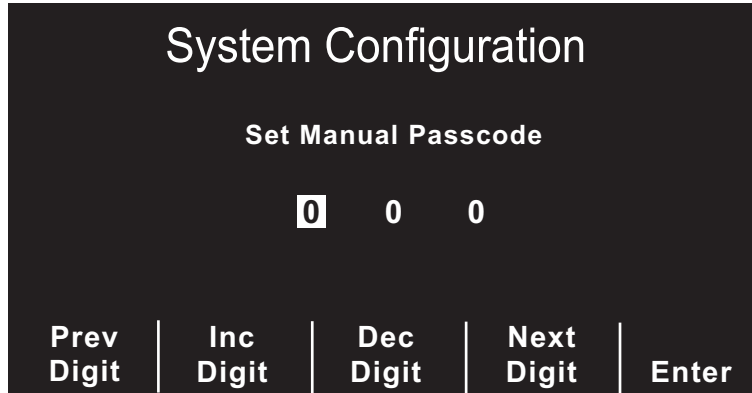


Pressing the **Prev Digit** softkey moves the highlight left to the previous entered digit.  
 Pressing the **Inc Digit** or **Dec Digit** softkeys increments or decrements the selected digit.  
 Pressing the **Next Digit** softkey moves the highlight right to the next digit to be entered.  
 Pressing the **Enter** softkey selects the values that have been entered and returns to main configuration menu.

## Configuration Mode Passcode

Sets an eight digit Configuration Passcode. Each digit entered can only be set between 0 (zero) and 3 (three) inclusive.

Pressing the **Change Value** softkey causes the following screen to display:

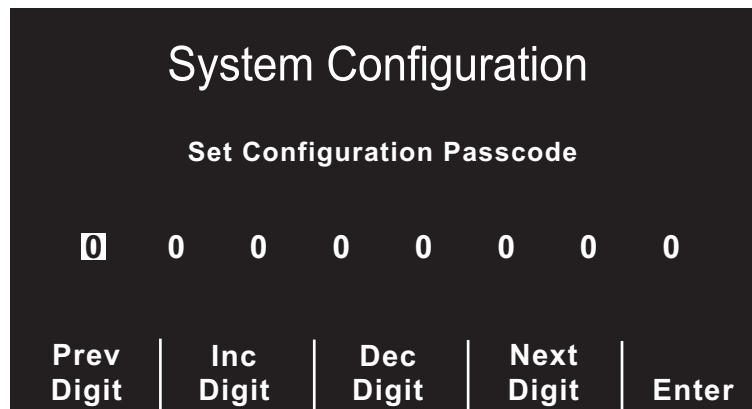

Pressing the **Prev Digit** softkey moves the highlight left to the previous entered digit.  
 Pressing the **Inc Digit** or **Dec Digit** softkeys increments or decrements the selected digit.  
 Pressing the **Next Digit** softkey moves the highlight right to the next digit to be entered.  
 Pressing the **Enter** softkey selects the values that have been entered and returns to main configuration menu.

## Energy Level: Shock 1, 2, 3

Selects the energy levels used for the 1<sup>st</sup>, 2<sup>nd</sup>, and 3<sup>rd</sup> consecutive shocks that the defibrillator delivers when:

- In semi-automatic or advisory mode and using pads
- In manual mode, using external paddles or pads, with Basic Auto Energy Escalation set to “Yes”

Energy level 1 is the energy level the unit defaults to when the power is turned on with external paddles or pads connected. For more information about the Basic Auto Energy Escalation setting, see the “Basic Auto Energy Escalation” section of this manual.

## Auto Analyze 3 Times

Sets the unit to automatically repeat ECG rhythm analyses after the delivery of shocks in the AED and Advisory Modes. When set to “Yes”, the unit automatically analyzes the patient’s ECG rhythm after the first and second shocks in each three-shock sequence. Following delivery of the third shock automatic reanalysis is halted and the “CHECK PULSE” message is displayed. This indicates that the three shocks sequence has been completed and the patient’s pulse should be checked. This function only operates when the “Auto Charge in Advisory Mode” configuration option is also enabled.

When set to “Only in Semi-auto,” the unit functions as described above only when it is in AED semi-auto mode. If the unit is changed to Manual mode when set to “Only in Semi-auto”, any automatic analysis/reanalysis sequencing terminates. If the **Analyze** button is pressed, the unit analyzes only once.

## "NO SHOCK ADVISED" Prompt

Changes the display message and the voice prompt issued after analysis of non-shockable rhythm.

## "CHECK PATIENT" Prompt

Changes the display message and the voice prompt issued when a shockable rhythm has been detected by background ECG analysis and Heart Rate Alarms are enabled, or the unit is operating in Semi-Automatic mode.

## Display DO CPR Message

Sets unit to display messages “IF NO PULSE” / “PERFORM CPR” alternately for one minute after each set of stacked shocks even if a perfusing rhythm is present.

ECG Rhythm analysis will restart automatically 70 seconds after delivery of the third shock in a three-analysis/shock sequence. The “CHECK PATIENT” message and voice prompt will also be inhibited for 70 seconds, following completion of a user activated analysis or discharge. This feature is operational only when the “Auto Charge in Advisory Mode” and “Auto Analyze 3 times” configuration options are enabled.

When set to “Only in Semi-Auto,” the unit functions as described above only when it is in AED semi-auto mode.

## Display ECG in Semi-Auto Mode

Sets the device to display or not display the patient’s ECG trace on the monitor while in semiautomatic mode.



## Store to Card in Auto Monitor Mode (AED models)

Sets unit to automatically store patient's ECG trace and voice data to the PCMCIA card when:

- ECG leads are attached to the patient
- MFE pads are not connected
- M Series CCT unit is on

If disabled, the M Series CCT unit will not record any data to the PCMCIA card unless MFE pads are connected to the patient.

## Auto Charge in Advisory Mode

When enabled and a shockable rhythm is detected after pressing the **ANALYZE** button, the unit will automatically charge the defibrillator to the pre-configured or user selected energy setting.

## Record to Card in Advisory Mode

Sets the unit to record ECG data (ECG only), ECG and voice data (Voice and ECG), or nothing (None) to the memory card.

## Enable Voice In Advisory Mode

Enables or disables voice prompts when operating in Advisory or Manual operating modes. If enabled, the voice prompts will sound at the appropriate time when in Advisory mode or Manual mode.

## Display Heart Rate in Semi-Auto Mode

Sets the device to display or not display the patient heart rate on the monitor while in semiautomatic mode.

## Display Lead on Power-Up

Allows the user to select any one of the following Leads to be displayed on power-up: Paddles/Pads, I, II, or III. When the unit powers up in semi-automatic mode, the option is ignored. The unit will power up in Lead II, and switch leads automatically between leads II and MFE for Auto Monitor and Auto Defib modes, respectively. When the unit enters Manual Mode (Manual Mode override sequence is completed), the lead switches to the setting specified in the "Display Lead on Power Up" option.

## Frequency Response

Selects the ECG filter bandwidth to be used during ECG monitoring. The frequency of the device can be set to either help reduce noise, help with the diagnostic response, or set as a default response.

## Enable 12-Lead in Semi-Auto Mode (AED with 12-Lead)

If set to "Yes," 12-lead analyses, transmissions and print-outs are available in semi-automatic (AED) Auto Monitor mode (ECG leads on the patient with MFE Pads disconnected), depending on other configuration option settings. If set to "No," 12-lead functions are available in Manual Mode only.

## QRS Volume on Power-Up

Turns the default QRS beeper volume to midrange or off following power up of the unit.

If set to "Off in Semi-Auto Mode," the QRS beeper volume turns off in semi-auto mode. If the unit is changed to manual mode, the beeper volume sets to midrange.

## Pace Rate Setting on Power-Up

Sets the default pace rate of the M Series CCT unit on power-up, between 30 ppm and 102 ppm (increments of 4 ppm).

## Display Implanted Pace Spikes

The M Series CCT is capable of detecting and indicating on the display pacemaker signals from a patient with an implantable pacemaker. Selecting “Yes” causes a pacemaker marker to be displayed on the ECG trace whenever implanted pacer stimuli are detected. The pacemaker spike detector is always active in ECG leads, independent of this configuration setting. This setting controls spike display only.

## Gain On Power-Up

Sets the default **SIZE** of the displayed ECG signal (AGC, 0.5 cm/mV, 1 cm/mV, 1.5 cm/mV, and 2 cm/mV) when the device is powered-up.

## Retain SYNC after Defib

Causes the unit to remain in SYNC mode after a synchronized cardioversion shock. The unit will remain in SYNC mode until the **SYNC** button is pressed again or the unit is switched out of Defib mode.

## Auto Generate Strips

Sets the strip chart recorder to automatically begin printing after a defibrillator discharge or whenever alarms are triggered while in manual mode. (Unit must be equipped with the optional strip chart recorder feature.)

## Print 3 Leads When Leads Sel

When enabled, the strip chart recorder will automatically print three (3) simultaneous leads of the patient ECG when Leads are selected and a five (5) lead or 12-lead ECG cable is in use.

## Report Memory Card Errors

Allows the user to disable memory card error messages and voice prompts.

If this option is set to "No", all error messages (i.e., “INSERT CARD”, “REPLACE CARD”, “CARD FULL”, and “CARD LOW < n MINUTES”) related to the use of the memory card will be suppressed. However, if the unit is in manual mode, or the unit is non-AED, the option is ignored when the "Record to Card in Advisory Mode" option is set to "None" (no data recording).

## Auto Analyze on Power Up (AED Semi-Automatic Mode)

When set to “Yes”, the unit will automatically begin an ECG rhythm analysis as soon as the device is turned on and pads are properly connected. An “ANALYZING ECG” message will be displayed for 5 seconds and a “STAND CLEAR” message is displayed and announced indicating the start of the analysis. If the Auto Analyze 3 Times option is enabled, the analysis will be the first of a three analysis sequence.

**Note:** The unit must be turned off for 10 seconds or more for the analysis to auto -start when the device is turned on.

## Auto Transmit After 12 Lead

When enabled, the unit will automatically enter the transmission setup screen following completion of a 12-lead analysis.

To fax using 2x6 format, this must be set to “Yes.”

## Fax/Communication Phone Numbers

Allows the user to pre-configure up to 24 locations and phone numbers and delete obsolete phone numbers. The M Series CCT uses the phone numbers to transmit data to a fax machine or Catalyst MUSE system following 12-lead analysis.

If you are transmitting to a Catalyst MUSE system, you can configure multiple phone numbers to access the same system. This will enable you to quickly try alternate numbers if you encounter difficulty transmitting to the Catalyst MUSE system. The system administrator provides these numbers when you initially set up access to the system. See Appendix B of the *12-Lead ECG Monitoring Operator's Guide* insert (part number 9650-0215-01) for more information.

You must also configure the Catalyst MUSE site and location identifiers before you can transmit to a Catalyst MUSE system. See "MUSE Site and Location" on page 21.

It is also recommended that you set up the M Series device identifier; see "Device Identifier" on page 10.


### Updating Fax/Communication Information

To transmit data to a fax machine or a Catalyst MUSE system, you must configure the following:

- The location of the medical facility to which data will be transmitted
- The phone number to which data will be transmitted
- The type of data to be transmitted ("Fax" if transmitting to a fax machine; "Data" if transmitting to Catalyst MUSE)

To update fax/MUSE communications phone numbers:

1. From the main System Configuration menu, press the **Next Item** softkey until "FAX/COMMUNICATION PHONE NUMBERS" highlights.
2. Press **Change Value** softkey. The Phone Numbers menu displays:

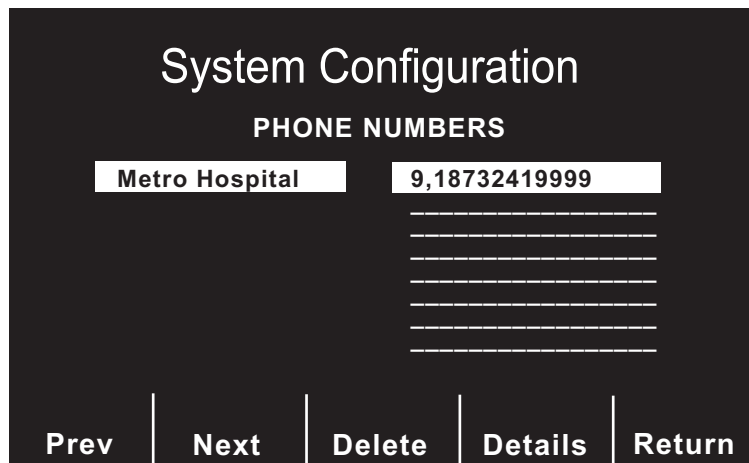

3. Press **Next** or **Prev** softkey until either:
  - The phone number to be changed highlights
  - An empty line highlights (if adding a new phone number)

4. Press **Details** softkey to select field. The Location and Phone Number menu displays:

The screenshot shows a dark background with white text. At the top, it says "System Configuration". Below that, there are two sections: "Location" with a small white box below it, and "Phone Number" with a dashed line below it. At the bottom, there are two options: "Fax or Data?" and "Fax". At the very bottom, there is a row of five softkeys: "Prev Letter", "Inc Letter", "Dec Letter", "Next Letter", and "Enter".

To update location of medical facility:

1. Press **Prev Letter** or **Next Letter** softkey to select letter.
2. Press **Inc Letter** or **Dec Letter** softkey to change value of letter.  
Repeat steps 1 and 2 until location has been entered.
3. Press **Enter** softkey to move cursor to first digit of phone number.

To update phone number:

1. Press **Prev Letter** or **Next Letter** softkey to select digit.
2. Press **Inc Letter** or **Dec Letter** softkey to change value of digit.  
Repeat steps 1 and 2 until entire phone number has been entered.
3. Press **Enter** softkey to move cursor to "Fax or Data?" field.

To specify type of phone number:

1. Press **Inc Letter** softkey to toggle between "Fax" and "Data." Select "Fax" to transmit to a fax machine.

This screenshot is similar to the previous one, but the "Fax or Data?" field is now set to "Fax". The rest of the interface, including the softkeys at the bottom, remains the same.

Select "Data" to transmit to a Catalyst MUSE system.

This screenshot is similar to the previous ones, but the "Fax or Data?" field is now set to "Data". The rest of the interface, including the softkeys at the bottom, remains the same.

2. Press **Enter** softkey.

## Deleting Fax/Communication Information

The following procedure deletes obsolete phone numbers from your M Series CCT unit.

To delete phone numbers:

1. From the main System Configuration menu, press **Next Item** softkey until “FAX/COMMUNICATION PHONE NUMBERS” highlights.
2. Press **Change Value** softkey.  
The Phone Numbers menu displays.
3. Press **Next** or **Prev** softkey to scroll to phone number.
4. Press **Delete** softkey to delete phone number.

## 12 Lead Analysis

This setting cannot be changed in the current version of software.

## 12 Lead Printout

When set to “Standard” the 12-lead 4x3 printout will use the standard lead format.

When set to “Cabrera” the 12-lead 4x3 printout will use the Cabrera-style lead format.

## Print 12 Lead Measurements

When set to “Yes” the 12SL™ Analysis will produce and print a measurements matrix including measurements for each lead. Global measurements will be produced and printed regardless of this setting. This setting affects the strip chart and summary report printouts as well as the fax transmission. When set to “Yes” any faxed 12-lead report will contain two pages. The second page of the report will contain the computed measurements matrix.

## 12 Lead Fax Format

When set to “4x3 Standard” the 12-lead format on the fax output is identical to the format on the strip chart, with the addition of a 10-second Lead II rhythm strip. (If the 12 LEAD PRINTOUT configuration option is set to “Cabrera” the fax will be Cabrera as well.)

When set to “2x6” the 12-lead format on the fax output will be configured for two columns of 6 leads, each with 5 seconds of data. The fax will not contain the 12SL interpretative statements or the 10-second Lead II rhythm strip. In addition, you must set “Auto Transmit After 12 Lead Analysis” to “Yes.”


The setting “4x3 Median Compl” is supported in software versions higher than 17.00. If “4x3 Median Compl” is selected, the 12-lead ECG waveforms on the strip chart, summary report, and fax will contain a single median complex beat computed by the GE Medical Systems Information Technologies 12SL analysis program.

## Print 2 Copies 12 Lead

When set to “Yes”, two copies of the 12-lead 4x3 report and 12SL analysis will print on the strip chart. This setting only affects 12-lead reports generated by pressing the **Acquire** softkey.

## Set Lead Groups

This setting allows the user to configure two custom lead groups as described in the 12-Lead ECG Monitoring insert (part number 9650-0215-01). Pressing the **Change Value** softkey will cause the following screen to appear:

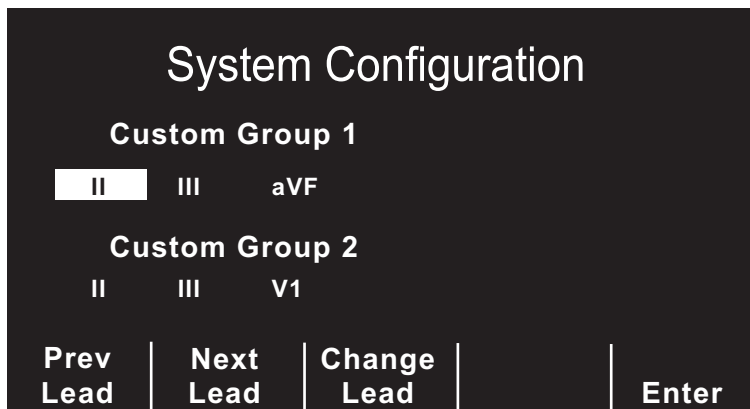

Pressing the **Prev Lead** softkey will select the previous lead.

Pressing the **Next Lead** softkey will select the next lead.

Pressing the **Change Lead** softkey will change the selected lead. Repeatedly pressing this key will cycle the selected lead through all twelve possibilities.

Pressing the **Enter** softkey will return to main configuration menu.

## 12 Lead Freq Response

When set to "0.05-150 Hz 4x3" the 12-Lead Filter Setting defaults to 0.05-150 Hz 4x3.

When set to "0.05-150 Hz Cont." the 12-Lead Filter Setting defaults to 0.05-150 Hz Cont.

When set to "0.05-40 Hz 4x3" the 12-Lead Filter Setting defaults to 0.05-40 Hz 4x3.

Refer to the *12-Lead ECG Monitoring Operator's Guide* insert (part number 9650-0215-01) for details on each setting.

## Lead Group Default

When set to "Standard" the 12-Lead Group Setting defaults to Standard.

When set to "Custom Group 1" the 12-Lead Group Setting defaults to the setting defined under Custom Group 1.

When set to "Custom Group 2" the 12-Lead Group Setting defaults to the setting defined under Custom Group 2.

Refer to the *12-Lead ECG Monitoring Operator's Guide* insert (part number 9650-0215-01) for details on each setting.

## Print 12 Lead Interpretation

When set to "Yes" the 12SL interpretative statements will print following the acquisition of a 12-lead ECG. This setting affects the strip chart and summary report printouts as well as the fax transmission.

## Defib Default to Pads

When set to "Yes" switching the unit to DEFIB mode will set the active lead to Pads/Paddles regardless of the configuration setting for "Display Lead on Power-Up."

## Cell Phone Type

This setting controls the active cellular phone profile for PCMCIA modem cards. It has no effect on landline functionality. Pressing the **Change Value** softkey selects different phone makes.

Select the phone make corresponding to your target cellular phone. For additional information on cellular phone compatibility contact The Supply Net, Inc. at [www.thesupplynet.com](http://www.thesupplynet.com) or (800) 826-0279.

## EtCO<sub>2</sub> Options

Pressing **Set/Review** causes a submenu to display, which allows review of or changes to the following capnography options:

- Default EtCO<sub>2</sub> Units: Sets EtCO<sub>2</sub> units to mmHg, kPa, or %
- Displayed Zoom Level: Sets the default scale setting for the displayed capnographic waveform. Does not affect the printed waveform or the waveform data stored on the data card.)
- Default EtCO<sub>2</sub> Comp Setting: Sets the default compensation for CO<sub>2</sub>

## Enable Leads Off

When set to “Yes” and ECG leads do not have proper contact with the patient, or the cable is not properly attached to the M Series CCT unit, the ECG LEADS OFF message displays and a dashed line displays in place of the ECG waveform. Yes is the default setting.

**Note:** When using 12 Lead or Pacer mode, the M Series CCT unit functions as if this configuration were set to “Yes.”

When set to “No” and ECG leads do not have proper contact with the patient or the cable is not properly attached to the M Series CCT unit, the POOR LEAD CONTACT message and the ECG waveform display.

**Caution:** Use care when interpreting ECG waveforms when the POOR LEAD CONTACT message displays, because the ECG waveform may include a significant amount of artifact. Whenever possible, reattach the leads before interpreting the waveform.

## Blood Pressure Units

Sets the NIBP and IBP unit of measurement to mmHg or kPa.

## NIBP Auto Interval Default

Sets the default for the amount of time between automatic measurements.

## Trigger NIBP Meas on BP Alarm

When set to “Yes,” the M Series unit initiates a single additional blood pressure measurement when any of the NIBP alarms trigger.


## Trigger NIBP Meas on HR Alarm

When set to “Yes,” the M Series CCT unit initiates a single blood pressure measurement when heart rate/pulse rate alarm triggers.

## NIBP First Inflate Pressure Default

Sets the default NIBP cuff inflation pressure (the pressure up to which the NIBP cuff inflates at the beginning of each measurement).

On M Series CCT units, you can set the NIBP cuff inflation pressure for three patient type modes: adult, pediatric and neonatal. With NIBP First Inflate Pressure Default highlighted and **Set/Review** selected, press the **Change Value** softkey. The following screen displays:

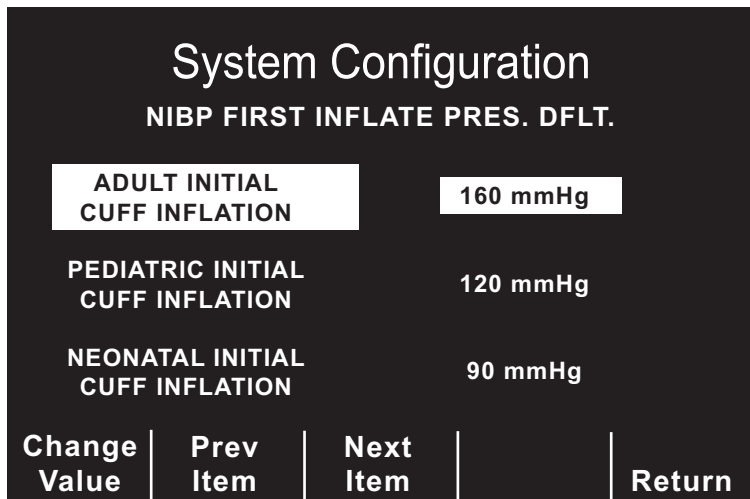

Pressing the **Change Value** softkey will scroll through the available inflation pressures for the highlighted parameter.

Pressing the **Prev Item** or **Next Item** softkeys moves the highlight to the previous or next parameter in the menu.

Pressing the **Return** softkey selects the values that have been entered and returns to the main configuration menu.

## Basic Auto Energy Escalation

When set to Yes, the unit automatically increments the defibrillation energy to the levels specified in Energy Level: Shock 1, 2, 3 after each of the first two shocks and displays the message “ENERGY INCREMENTED,” when both of the following are true:

- The defibrillator is in Manual mode
- The defibrillator has external paddles or pads attached

Manually changing the energy level outside the pre-programmed sequence and delivering a shock disables this function until the unit is turned off for more than 10 seconds, then turned back on.

This option increments the energy level regardless of Shock Advisory activation or status. It does not work with internal handles.

For more information about the energy level settings, see the “Energy Level: Shock 1, 2, 3” section of this manual.




## MUSE Site and Location

Sets the Catalyst MUSE site and location identifiers needed to transmit 12-lead ECG data to a Catalyst MUSE system. The medical facility’s Catalyst MUSE system administrator provides these numbers when you initially set up access to the system. The Catalyst MUSE site has a value between 01 and 32, and the location has a value between 001 and 600.

**Note:** You must also configure the Catalyst MUSE communications phone number. See the “Fax/Communication Phone Numbers” section of this guide.

To set the Catalyst MUSE site and location:

1. From the main System Configuration menu, press **Next Item** softkey until “MUSE SITE AND LOCATION” highlights.

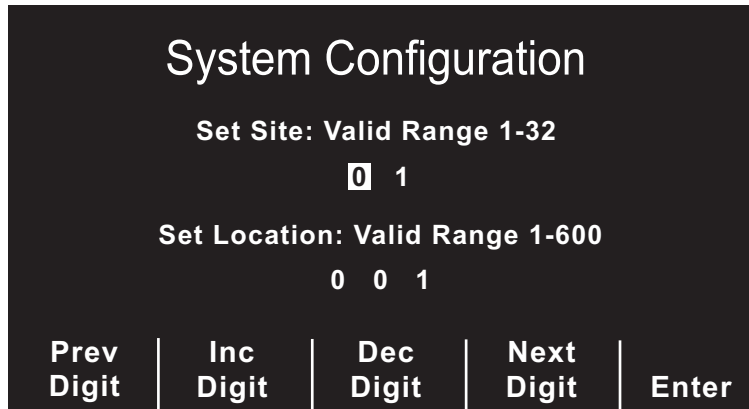

2. Press the **Change Value** softkey to display the MUSE Site and Location menu:
3. Press **Prev Digit** or **Next Digit** softkey to select digit and move to previous or next digit.
4. Press **Inc Digit** or **Dec Digit** softkey to change value of digit.  
Repeat steps 3 and 4 until site has been entered.
5. Press **Enter** softkey to move cursor to first digit of location.  
Repeat steps 3 and 4 until location has been entered.
6. Press **Enter** softkey to accept site and location values and return to main configuration menu.

## 12 Lead Secs Per Lead

Sets the number of seconds of ECG data printed for each lead on the 12-lead 4x3 strip. This setting only affects 12-lead reports generated by pressing the **Acquire** softkey.

## 12 Lead Print Speed

Sets the print speed of the 12-lead 4x3 reports to either 25mm/second or 50mm/second. This setting only affects 12-lead reports generated by pressing the **Acquire** softkey.

## Upload Baud Rate

Sets the upload transmission rate to 9600 bps, 38400 bps, or 115200 bps. This option is only applicable to units equipped with Bluetooth or RS-232 communications hardware.

## Trending Report Zoom Level

Sets the power up resolution for the displayed trend report. Trend data values for all physiological parameters are simultaneously sampled by the M Series CCT and stored in memory once each minute.

When displaying this data in tabular format on the M Series CCT screen, however, the user can choose to view all or only a subset of the stored trend data. The factory default zoom level causes all one-minute data samples to be displayed when viewing the trend table. If lower resolution is desired, this configuration option can be set to display only the samples taken

every 5, 10, 15, 30, or 60 minutes. All data samples are printed on the strip chart regardless of this setting.

## Trending Enabled

When set to “Yes”, the M Series CCT unit will print and display trend reports as long as the appropriate hardware is installed. Defaults to “No” for units not equipped with the appropriate hardware.

## AED Defib Ready Hold Time (AED Semi-Automatic Mode)

Sets the number of seconds the unit sounds a charge ready tone, indicating that the defibrillator is charged and ready to use. The charge ready tone stops when you discharge the defibrillator or when the hold time for the charge ready tone has elapsed. You can set the hold time for the charge ready tone to 15 seconds or 60 seconds. The default is 15 seconds.

## Battery Type Used

Allows the user to select between the PD 4410 battery (default) and the newer XL battery. To maximize battery life, it is important that the setting reflect the actual battery in use. Each battery type has a different threshold for “low battery” and “shutdown”, and has a different Amp-hour rating.

### **WARNINGS:**

If an XL battery is installed in an M Series CCT, the user must ensure that the “BATTERY TYPE USED” is set to “XL Battery.” Otherwise, the M Series CCT may abnormally terminate battery charging prior to completion when charging a depleted or nearly depleted XL battery. If battery charging is abnormally terminated, the M Series CCT battery will toggle the “CHARGER ON” light between amber and green, and, when M Series CCT power is applied, it will display the message “BATT. OVERCHARGE” on the screen for approximately 30 seconds.

Note that if battery charging is abnormally terminated because the “BATTERY TYPE USED” has been incorrectly set to “PD 4410”, the XL battery can still be charged to completion by cycling M Series CCT power from “off” to “on” and then back to “off” to clear this condition.

Incorrectly setting “BATTERY TYPE USED” to “PD 4410” when an XL battery is installed will cause the “LOW BATTERY” warning message to be displayed on the M Series CCT screen prematurely.

Incorrectly setting the selection to “XL battery” when a PD 4410 battery is installed will inappropriately delay issuance of the “LOW BATTERY” warning and severely limit the number of defibrillation shocks which can be delivered between the onset of the “LOW BATTERY” warning message and the M Series CCT shutdown.

## Alarm Limits at Power Up

Allows review of, or changes to, the power-up default alarm state and alarm limits for each physiological parameter installed in the unit.

With Alarm Limits At Power Up highlighted and Set/Review selected, press the **Change Value** softkey. The following screen displays:

On M Series CCT units, the following screen displays:

System Configuration				
SET ALARM DEFAULTS				
Parameter	State	Low	High	
ECG HR	ENABLE	30	150	
SpO2	ENABLE	85	100	
NIBP SYS	ENABLE	90	160	
NIBP DIA	ENABLE	50	110	
NIBP MEAN	ENABLE	60	130	
P1 SYS	ENABLE	90	160	
P1 DIA	ENABLE	50	110	
P1 MEAN	ENABLE	60	130	
Page 1 of 4				
Select Param	Inc >	Dec <	Next Field	Return

Pressing the **Select Param** softkey will scroll the highlight through the available parameters. Pressing the **Inc** or **Dec** softkeys increments or decrements the value in the highlighted field. Pressing the **Next Field** softkey moves the highlight to the next field in the table. Pressing the **Return** softkey selects the values that have been entered and returns to the main configuration menu.

## Alarms Active at Power Up

When set to “Yes”, alarms are automatically active at power up. The setting applies to all alarms enabled by the Alarm Limits At Power Up option.


## Low Battery Tone

Sets the interval between Low Battery warning beeps to either 1 minute or 5 minutes. The Low Battery warning message is displayed once per minute, regardless of this setting.

## Clock Synchronization

Allows the user to set clock synchronization parameters on the M Series CCT unit.

Highlight the Clock Synchronization option and press the **Change Value** softkey to display the following screen:

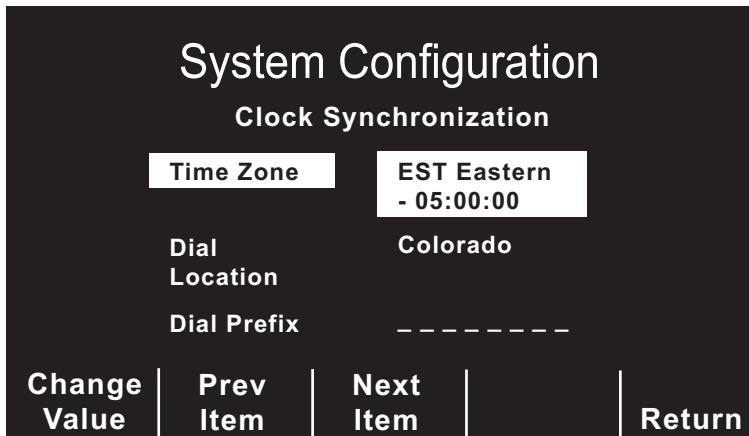

Press the **Change Value** softkey to scroll through the available values for each parameter.

Press the **Next Item** softkey to move to the next parameter in the table.

Press the **Return** softkey to save the values entered and return to the main configuration menu.

You can set the values for the following Clock Synchronization options:

- **Time Zone:** Selects the time zone in which the M Series CCT will be used. Choices include: Azores, Middle Atlantic, S. America Eastern, AST Atlantic, EST Eastern, CST Central, MST Mountain, PST Pacific, AKST Alaska, HST Hawaiian, Samoa, Dateline, GMT Greenwich, CET Central Europe, EET Eastern Europe, MSK Moscow, Arabian, Afghanistan, West Asia, Central Asia, Southeast Asia, AWST Australian West, Korean, ACST Australian Cent, AEST Australian East, Central Pacific, or New Zealand
- **Dial Location:** Sets the National Institute of Science and Technology (NIST) dialing location to either Hawaii or Colorado. This is the location called by the M Series CCT to obtain accurate time information for setting its internal clock.
- **Prefix:** Affixes up to eight user selectable digits or special characters (+, &, !, \*, or comma) to the start of the dialing location phone number.

## Daylight Savings

When set to “Yes” and the user dials the selected NIST location, the unit will automatically adjust the time for Daylight Savings based on the assumption that DST starts on the first Sunday in April at 1:59:59 AM (setting the time ahead by one hour) and ends on the last Sunday in October at 1:59:59 AM (setting the time back by one hour).

## Card Capacity Message Enabled

When set to “Yes” and data recording is enabled with a data card present, the M Series CCT will display “CARD LOW” messages when the remaining storage capacity of the card reaches 30, 15, and 5 minutes.

To enable this option, the “Report Memory Card Errors” option must be set to “Yes”; otherwise, the option is ignored.

## Erase Card Prompt at Pwr Off

Sets the M Series CCT unit to allow the user to erase the data card at power-off if the card contains only self test, and optionally, synchronized cardioversion data. To enable this option,

the “Allow Card Erase” option must be set to “Yes”, and the “Record to Card in Advisory Mode” option must be set to “ECG only” or “Voice and ECG”; otherwise, this option is ignored. If set to “Yes”, the unit will display the message “Erase Card -- Yes/No?” when the user turns the Selector Switch to the OFF position and the following criteria are met:

1. A data card is installed in the M Series CCT unit.
2. The unit has NOT performed a defibrillation, other than a self test, since power-up.  
**Note:** The M Series CCT unit can also be configured to display the “Erase Card -- Yes/No?” prompt if the card contains only synchronized cardioversion data. See the “Sync Defib Excluded” option for more information.
3. The unit has NOT delivered a pacing current greater than 25 mA since power-up.

If the user selects “Yes” at the prompt within 15 seconds, the card will be erased and the unit will subsequently shut off. If the user selects “No”, or 15 seconds is exceeded without a response, the unit will shut off without erasing the card.

## Remove Card Prompt at Pwr Off

Sets the M Series CCT unit to prompt the user to remove the data card at power-off if the card contains any information other than self test, and optionally, synchronized cardioversion data.

If set to “Yes”, the unit will display the message “Remove Card” when the user turns the Selector Switch to the OFF position and the following criteria are met:

1. A data card is installed in the M Series CCT unit.
2. The unit HAS performed a defibrillation, other than a self test, since power-up.  
**Note:** The M Series CCT unit can be configured NOT to display the “Remove Card” prompt if the card contains only synchronized cardioversion data. See the “Sync Defib Excluded” option for more information.
3. The unit HAS delivered a pacing current greater than 25 mA since power-up.

When this option is enabled, the “Remove Card” message will be displayed for 10 seconds, then the M Series CCT unit will shut off.

## Sync Defib Excluded

Allows the exclusion of synchronized cardioversion from the criterion of a defibrillation event, for the purposes of the “Erase Card Prompt at Pwr Off” and “Remove Card Prompt at Pwr Off” configuration settings.

When set to “Yes”, if the M Series CCT unit has only discharged self test or synchronized cardioversion defibrillations and has not delivered pacing current greater than 25 mA, the unit displays the “Erase Card -- Yes/No” prompt at power-off (if so configured). Otherwise, the unit displays the “Remove Card” prompt at power-off (if so configured).

## PTCA Settings

Allows review of, or changes to Percutaneous Transluminal Coronary Angioplasty (PTCA) settings if the M Series CCT includes the 12-Lead Reperfusion Algorithm option.

With PTCA Settings highlighted and Set/Review selected, press the **Change Value** softkey to set the following parameters:

- **Message Enabled:** Determines whether or not the M Series CCT unit prints a PTCA message (if appropriate).
- **Threshold:** Sets the threshold, in mV, for issuing the PTCA message. Only available if the Message Enabled option is set to “Yes”.

## Power Up Patient Type (NIBP)

Sets the patient type (adult, pediatric, or neonatal) for NIBP measurements when the unit is powered up.


## Temperature Units

Sets the temperature measurement unit to °F or °C.

## Display Colors

Allows the user to set individual display colors for parameters displayed on the M Series CCT monitor.

Highlight the Display Colors option and press the **Change Value** softkey to display the following screen:

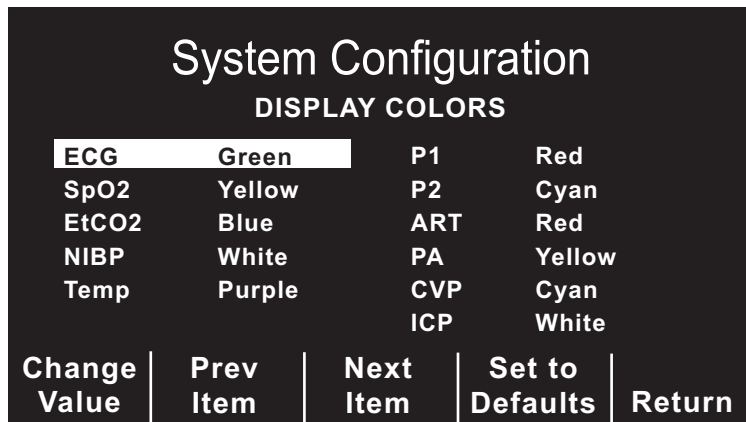

Press the **Change Value** softkey to scroll through the available colors for each parameter.


Press the **Next Item** softkey to move to the next parameter in the table.

Press the **Return** softkey to save the values entered and return to the main configuration menu.

## Power Up Trace Selection

Allows the user to set the parameter displayed on the second and third traces respectively on the M Series CCT monitor when the unit is powered up.

Highlight the Power Up Trace Selection option and press the **Change Value** softkey to display the following screen:

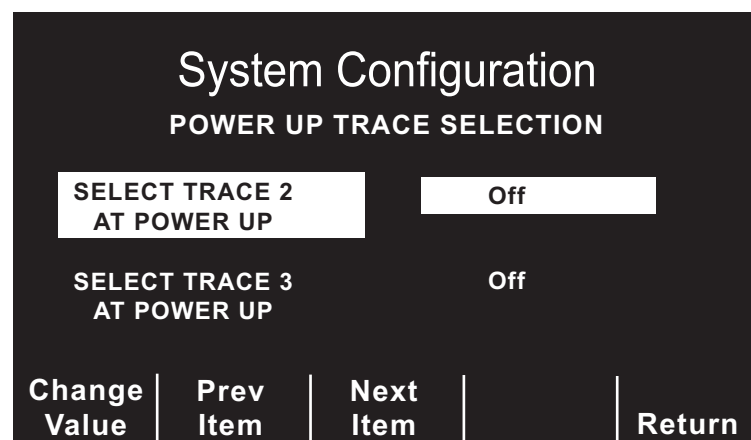

Press the **Change Value** softkey to scroll through the available parameters for each Trace selection.

Press the **Next Item** softkey to move to the next Trace selection in the table.

Press the **Return** softkey to save the values entered and return to the main configuration menu.

### **IBP1 Label at Power Up**

Sets the label assigned to the first IBP channel when the unit is powered up.

### **IBP2 Label at Power Up**

Sets the label assigned to the second IBP channel when the unit is powered up.

### **IBP1 Range at Power Up**

Sets the default display scale (in mmHg) for the waveform amplitude of the first IBP channel at power-up.

### **IBP2 Range at Power Up**

Sets the default display scale (in mmHg) for the waveform amplitude of the second IBP channel at power-up.

(This page has intentionally been left blank.)



# Configuration Option Tables

The following tables list the M Series CCT option and their possible values.

An asterisk (“\*”) refers to features that are currently not implemented on the M Series CCT. These features will only operate in accordance with their pre-configured default settings (listed below).

## General

Feature	Options	Default
Selected Language	English, Other	English
Notch Filter (ECG)	50, 60 Hz	60 Hz
Allow Card Erase	Yes/No	Yes
Display Elapsed Time	Yes/No	AED: Yes Manual Advisory: No
* Voice Markers Enabled	Yes/No	No
Set Report Restart Delays	Set/Review	Set/Review
Summary Report Restart Delay	5, 15, 30, 90 minutes, 1.5 days	15 minutes
Trend Report Restart Delay	5, 15, 30, 90 minutes, 1.5 days	15 minutes
Print 12 Lead 4x3	None, 1 Copy, 2 Copies	None
* Auto Self-Test Time-out Period	No Test, 7, 8, 9, 10, 11, 12, 13, 14, 15, 16, 17, 18, 19, 20, 21, 22, 23, 24, 25, 26, 27, 28, 29, 30 days	No test
Async Softkey in Pace Mode	Yes/No	Yes
Code Markers (Defib Code Markers, Monitor Code Markers, Pace Code Markers)	Config/Review	Config/Review
Device Identifier	0 through 9, A through Z, blank, and “_” inclusive for each digit	0 0 0 0 0 0 0 0 0 0
Manual Mode Passcode	0 through 3 inclusive, for each digit	0 0 0
Configuration Mode Passcode	0 through 3 inclusive, for each digit	0 0 0 0 0 0 0 0
Display Lead on Power-Up	Paddles/Pads, I, II, or III	Paddles/Pads
Frequency Response	0.5 - 21 Hz, 0.5 - 27 Hz, 1 - 21 Hz	0.5 - 21 Hz
QRS Volume on Power-Up	Midrange, OFF	Midrange
Pace Rate Setting on Power-Up	30 through 102 ppm (increments of 4 ppm)	70 ppm
Displayed Implanted Pace Spikes	Yes/No	Yes
Gain on Power-Up (ECG)	AGC, 0.5 cm/mV, 1 cm/mV, 1.5 cm/mV, 2 cm/mV	AGC
Report Memory Card Errors	Yes/No	Yes
Enable Leads Off	Yes/No	Yes
Upload Baud Rate	9600 bps, 38400 bps, 115200 bps	38400 bps
Trending Report Zoom Level	All, 5 min., 10 min., 15 min., 30 min., 60 min.	All
Trending Enabled	Yes/No	Yes
Battery Type Used	PD 4410, XL Battery	PD 4410
Alarm Limits at Power-Up	Set/Review	Set/Review
Alarms Active at Power-Up	Yes/No	No
Low Battery Tone	1, 5 minutes	1 minute

## M SERIES CCT CONFIGURATION GUIDE

---

<b>Feature</b>	<b>Options</b>	<b>Default</b>
Clock Synchronization	Set/Review	Set/Review
Time Zone	Azores, Middle Atlantic, S. America Eastern, AST Atlantic, EST Eastern, CST Central, MST Mountain, PST Pacific, AKST Alaska, HST Hawaiian, Samoa, Dateline, GMT Greenwich, CET Central Europe, EET Eastern Europe, MSK Moscow, Arabian, Afghanistan, West Asia, Central Asia, Southeast Asia, AWST Australian West, Korean, ACST Australian Cent, AEST Australian East, Central Pacific, New Zealand	EST Eastern
Dial Location	Hawaii, Colorado	Colorado
Dial Prefix	8 digits	(none)
Daylight Savings	Yes/No	No
Card Capacity Message Enabled	Yes/No	No
Erase Card Prompt at Pwr Off	Yes/No	No
Remove Card Prompt at Pwr Off	Yes/No	No
Sync Defib Excluded	Yes/No	No

## Advisory and Semi-automatic Modes

Feature	Options	Default
Energy Level: Shock 1	Damped Sine Wave (DSW): 1-10, 15, 20, 30, 50, 75, 100, 150, 200, 300, 360 J  Biphasic: 1-10, 15, 20, 30, 50, 75, 100, 120, 150, 200 J	Manual DSW: 200 J Manual Biphasic: 120 J  AED DSW: 200 J AED Biphasic: 120 J
Energy Level: Shock 2	Damped Sine Wave (DSW): 1-10, 15, 20, 30, 50, 75, 100, 150, 200, 300, 360 J  Biphasic: 1-10, 15, 20, 30, 50, 75, 100, 120, 150, 200 J	Manual DSW: 300 J Manual Biphasic: 150 J  AED DSW: 200 J AED Biphasic: 120 J
Energy Level: Shock 3	Damped Sine Wave (DSW): 1-10, 15, 20, 30, 50, 75, 100, 150, 200, 300, 360 J  Biphasic: 1-10, 15, 20, 30, 50, 75, 100, 120, 150, 200 J	Manual DSW: 360 J Manual Biphasic: 200 J  AED DSW: 360 J AED Biphasic: 200 J
Auto Analyze 3 Times	Yes, No, Only in Semi-Auto	Manual: No AED: Only in Semi-Auto
"NO SHOCK ADVISED" Prompt	"NO SHOCK ADVISED", "CHECK PULSE", BOTH	"NO SHOCK ADVISED"
"CHECK PATIENT" Prompt	"CHECK PATIENT", "PRESS ANALYZE"	"CHECK PATIENT"
Display "DO CPR" Message	Yes, No, Only in Semi-Auto	Manual: No AED: Only in Semi-Auto
Display ECG in Semi-Auto Mode	Yes/No	Yes
Store to Card in Auto Monitor Mode	Yes/No	Yes
Auto Charge in Advisory Mode	Yes/No	Yes
Record to Card in Advisory Mode	None, ECG Only, Voice and ECG	None
Enable Voice in Advisory Mode	Yes/No	No
Display Heart Rate in Semi-Auto Mode	Yes/No	No
Auto-Analyze on Power Up	Yes/No	No
Enable 12-Lead in Semi-Auto Mode	Yes/No	Yes
AED Defib Ready Hold Time	15 seconds, 60 seconds	15 seconds

## Manual Mode

Feature	Options	Default
Auto Generate Strips	Yes/No	Yes
Retain SYNC after Defib	Yes/No	No
Print 3 Leads When Leads Sel.	Yes/No	No
Auto Transmit After 12 Lead	Yes/No	No
Fax/Communication Phone Numbers	Set/Review	Set/Review
12 Lead Analysis	Current 10 sec	Current 10 sec
12 Lead Printout	Standard/Cabrera	Standard
Print 12 Lead Measurements	Yes/No	No
12 Lead Fax Format	4x3 Standard, 4x3 Median Compl, 2x6	4x3 Standard
Print 2 Copies 12 Ld Analysis	Yes/No	No
Set Lead Groups	Set/Review	Set/Review
Custom Group 1 (3 leads)	For each lead: I, II, III, aVR, aVL, aVF, V1, V2, V3, V4,V5,V6	Custom Group 1: II III aVF
Custom Group 2 (3 leads)		Custom Group 2: II III V1
12 Lead Freq Response	0.05 - 150 Hz 4x3, 0.05 - 150 Hz Cont., 0.05 - 40 Hz 4x3	0.05 - 150 Hz 4x3
Lead Group Default	Standard, Custom Group 1, Custom Group 2	Standard
Print 12 Lead Interpretation	Yes/No	Yes
Defib Default to Pads	Yes/No	No
Cell Phone Type	Motorola, Nokia, NEC, OKI, Sony, Ericsson	Motorola
Basic Auto Energy Escalation	Yes/No	No
MUSE Site and Location	Site: 01 - 32 Location: 001 - 600	Site: 01 Location: 001
12 Lead Secs Per Lead	2.5, 5.0, 7.5, 10.0 seconds	2.5 seconds
12 Lead Print Speed	25 mm/sec, 50 mm/sec	25 mm/sec

## EtCO<sub>2</sub> Option

Feature	Options	Default
EtCO <sub>2</sub> Options	Set/Review	Set/Review
Default EtCO <sub>2</sub> Units	mmHg, kPa, %	mmHg
Displayed Zoom Level	(in mmHg): 0-12.5, 0-25, 0-50, 0-75, 0-100 (in kPa): 0-1.7, 0-3.3, 0-6.6, 0-10.0, 0-13.3	0-50 mmHg 0-6.6 kPa
Default EtCO <sub>2</sub> Comp Setting	None, O2, N2O, O2 & N2O	None

## NIBP Option

Feature	Options	Default
NIBP Units (Blood Pressure Units on CCT models)	mmHg, kPa	mmHg
NIBP Auto Interval Default	2.5, 5, 10, 15, 20, 30, 45, 60, 90, 120 minutes	30 minutes
Trigger NIBP Meas on BP Alarm	Yes/No	Yes
Trigger NIBP Meas on HR Alarm	Yes/No	No
NIBP First Inflate Pressure Default	Set/Review	Set/Review
Adult Initial Cuff Inflation (CCT models)	(in mmHg): 120, 140, 160, 180, 200, 220, 240, 260 (in kPa): 16.0, 18.7, 21.3, 24.0, 26.7, 29.3, 32.0, 34.7	160 mmHg
Pediatric Initial Cuff Inflation	(in mmHg): 80, 90, 100, 110, 120, 130, 140, 150 (in kPa): 10.7, 12.0, 13.3, 14.7, 16.0, 17.3, 18.7 20.0	120 mmHg
Neonatal Initial Cuff Inflation	(in mmHg): 60, 70, 80, 90, 100, 110, 120, 130 (in kPa): 8.0, 9.3, 10.7, 12.0, 13.3, 14.7, 16.0, 17.3	90 mmHg
Power Up Patient Type (NIBP)	Adult, Pediatric, Neonatal	Adult

## PTCA Settings Option

Feature	Options	Default
Message Enabled	Yes/No	Yes
Threshold	(in mV): 0.8, 0.9, 1.0, 1.1, 1.2, 1.3, 1.4, 1.5, 1.6, 1.7, 1.8, 1.9, 2.0	1.5 mV

## Expanded Display Option

Feature	Options	Default
Display Colors	Set/Review	Set/Review
Power Up Trace Selection	Set/Review	Set/Review

## IBP Option

Feature	Options	Default
Blood Pressure Units	mmHg, kPa	mmHg
IBP1 Label at Power Up	P1, ART, PA, CVP, ICP	P1
IBP2 Label at Power Up	P2, ART, PA, CVP, ICP	P2
IBP1 Range at Power Up	(in mmHg): 0-300, 0-180, 0-120, 0-60, 0-30 (in kPa): 0-40, 0-24, 0-16, 0-8, 0-4	0-300 mmHg
IBP2 Range at Power Up	(in mmHg): 0-300, 0-180, 0-120, 0-60, 0-30 (in kPa): 0-40, 0-24, 0-16, 0-8, 0-4	0-120 mmHg

## Temperature Option

Feature	Options	Default
Temperature Units	°C, °F	°F